


How to: Use the Veterinary Web Portal

Notes

- **All veterinarians** at the **same clinic** can see all the patients added to the web portal
- When an **account is created**, the option is given to send email **invites** to other **veterinarians** at the **practice**, but veterinarians can be added/removed at any time by logging into an active account.

1 Sign in or Sign up

Go to www.GluCurve.com, **returning veterinarians** can enter username and password then click log in

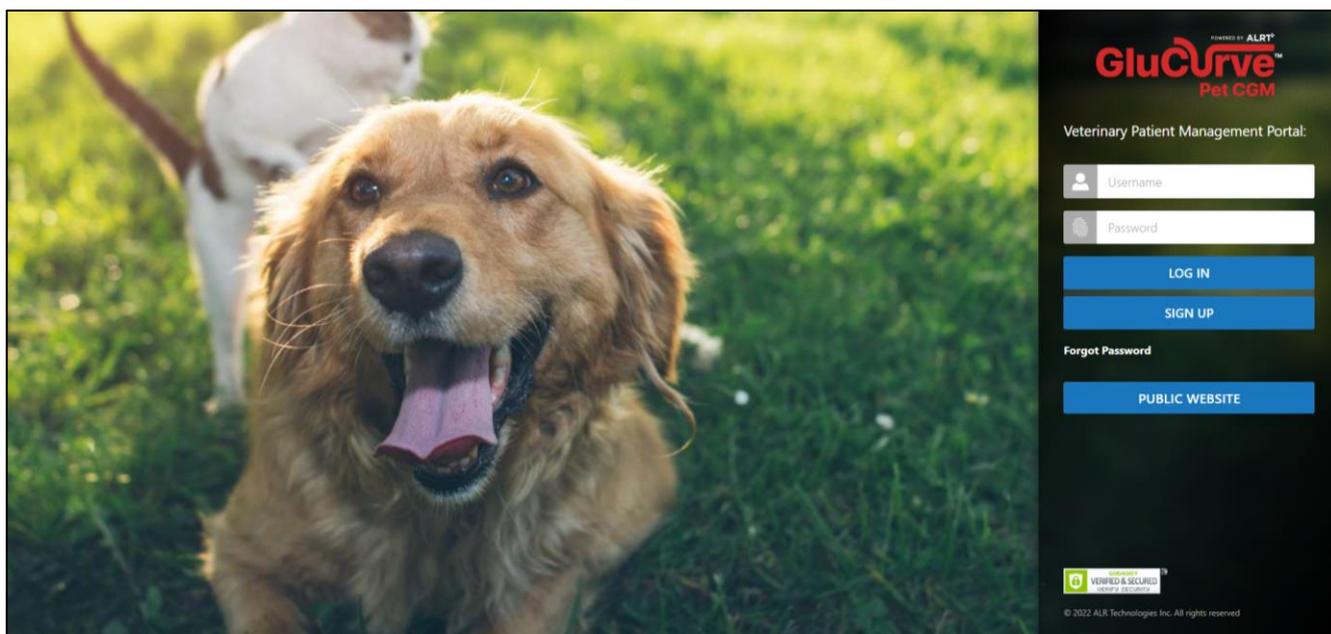

If this is your **clinic's first time** using a GluCurve Pet CGM, click sign up and create a free account

1. On the sign up page, enter the information for **each veterinarian** at your **practice** to give each doctor their own **access**

How to: Use the Veterinary Web Portal

2 List of Patients

After signing in you will be taken to your clinic's list of patients

1. Return to this list at any time by clicking Patient List.
2. Click here to access your account or log out
3. Click here to add a new GluCurve Pet CGM patient
4. Click any arrow tab to sort the list
5. Click on pet name or owners last name to view patient

The screenshot shows the 'LIST OF PATIENTS' interface. At the top right, there are navigation links for 'PATIENT LIST' (1) and 'GLUCURVE TRAINING' (2). Below these is a search bar and an 'Add new patient' button (3). The main table has columns for 'PET'S NAME', 'PET OWNER'S LAST NAME', 'LAST GLUCOSE CURVE', 'TYPE', and 'ACTION'. Red arrows (4) point to the sort arrows in the table headers. Red arrows (5) point to individual pet names in the table. A table with 13 rows of patient data is shown below.

PET'S NAME	PET OWNER'S LAST NAME	LAST GLUCOSE CURVE	TYPE	ACTION
app test	test 1	11/8/2022 5:50:34 PM	Dog	[trash icon]
abigail	Test 11	9/22/2022 2:50:03 PM	Cat	[trash icon]
jack	test10	6/23/2022 12:29:36 AM	Dog	[trash icon]
nacho	test 8	3/8/2022 4:01:48 PM	Cat	[trash icon]
dillinger	test1	3/4/2022 10:03:13 AM	Dog	[trash icon]
kenwood	test5	3/3/2022 10:33:05 AM	Cat	[trash icon]
sadie	test6	3/2/2022 9:33:37 AM	Cat	[trash icon]
diamond	test 9	3/1/2022 12:17:24 PM	Dog	[trash icon]
rudolf	test3	2/23/2022 5:30:37 PM	Dog	[trash icon]
goose	test7	2/19/2022 8:31:20 AM	Cat	[trash icon]
herman	test4	2/18/2022 10:14:28 PM	Cat	[trash icon]
skeletor	test2	2/18/2022 1:31:26 PM	Dog	[trash icon]
rudolf	test3		Dog (Pending registration)	[trash icon] [edit icon]

3 Glucose Curve

After clicking a patient name you will be taken to the glucose curve tab for that patient

1. Click each individual dates bar to see the individual day's glucose curve
2. Click the box on each day's glucose curve you want to compare and overlay to display in the top graph
3. Click show data points to see each individual 3 minute glucose reading for the day


The screenshot shows the 'GLUCOSE CURVE' page for 'DILLINGER TEST1 (DOG)'. It features a line graph showing glucose levels over time. A legend indicates 'Insulin' (red circle), 'Meal' (blue circle), and 'Both' (black circle). Below the graph, there are 'Check boxes to compare curves' for three dates: 2022/03/04 (2), 2022/03/03 (1), and 2022/03/02 (3). Each date has a 'CLICK HERE TO VIEW CURVE' link and a 'SHOW DATA POINTS' button. A red arrow on the left points to the patient name in the header.

How to: Use the Veterinary Web Portal

4 Insulin Calculator


Click the Insulin Calculator tab for a list of insulins, calculators, guidelines, and flowcharts.

1. Click the calculator icon to pull up an insulin dose calculator
2. Click the document icon to pull up suggested dosing guidelines
3. Click flow chart icon to pull up the Manufacturer or AAHA dosing flow chart

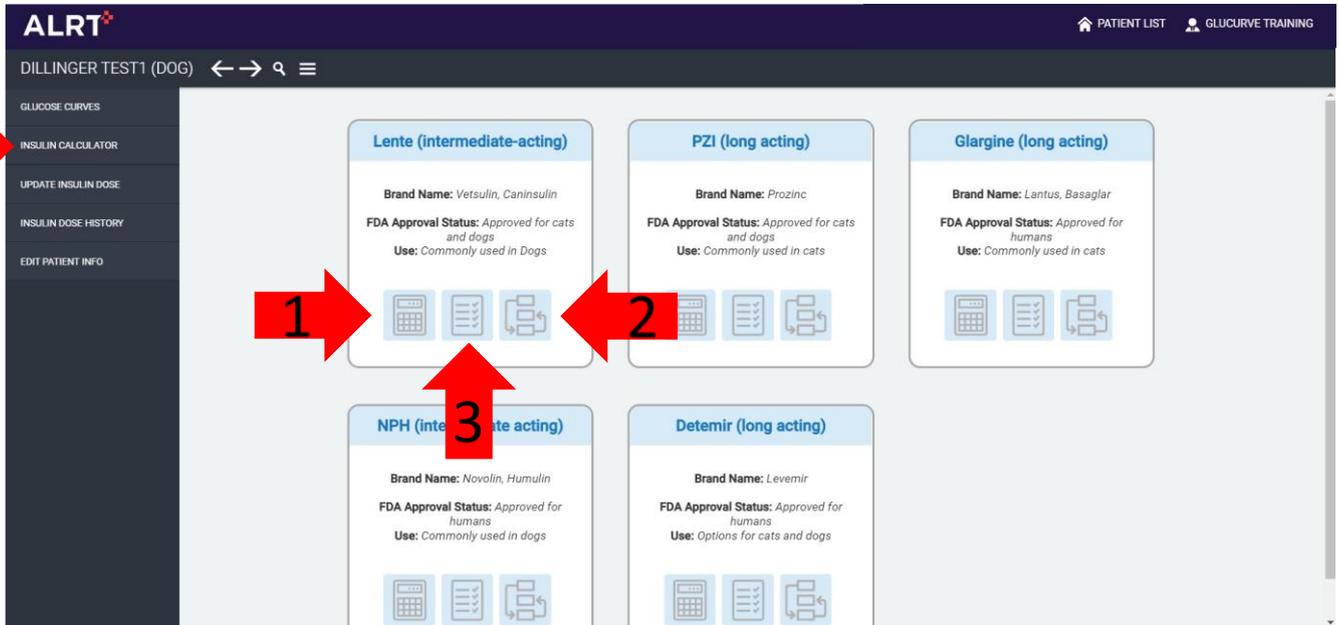

5 Update Insulin Dose

Click the update insulin dose tab and fill out the information whenever you switch doses for tracking purposes.

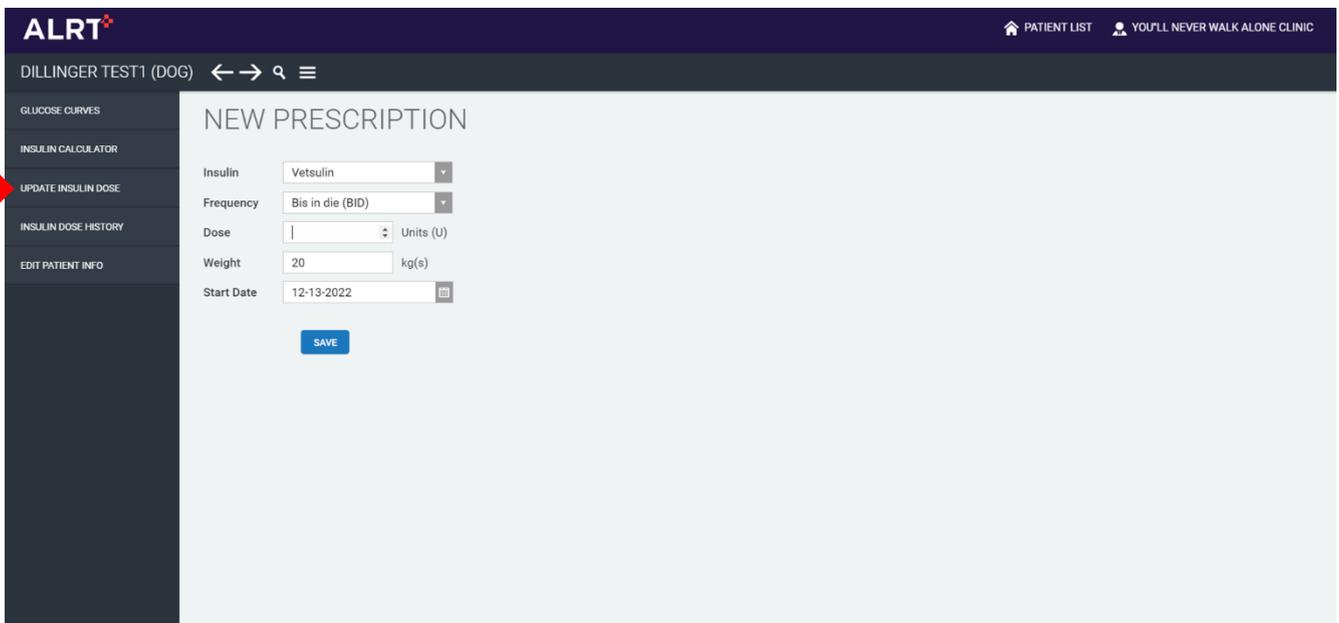

6 Insulin Dose History

Click the Insulin Dose History tab to display the insulin dose history for this patient

ALRT

PATIENT LIST GLUCURVE TRAINING

DILLINGER TEST1 (DOG) ← → 🔍 ☰

GLUCOSE CURVES

INSULIN CALCULATOR

UPDATE INSULIN DOSE

INSULIN DOSE HISTORY

EDIT PATIENT INFO

PRESCRIPTION HISTORY

EXPORT TO EXCEL

DATE PRESCRIBED	PET WEIGHT	INSULIN	UNITS	FREQUENCY	ACTION
2022/11/21	7 kg	Vetsulin	0.4 units	BID	

SHOWING 1 TO 1 OF 1 ENTRIES

7 Edit Patient Info

Click the Edit Patient Info to verify to:

- Verify pet owners username used to log into the GluCurve app and reset their password if needed
- Edit pet owner email or phone number
- Edit patient name
- Click Devices to see a list of Bluetooth Transmitters used for this patient and deactivate if needed

ALRT

PATIENT LIST GLUCURVE TRAINING

DILLINGER TEST1 (DOG) ← → 🔍 ☰

GLUCOSE CURVES

INSULIN CALCULATOR

UPDATE INSULIN DOSE

INSULIN DOSE HISTORY

EDIT PATIENT INFO

Profile Devices

LOGIN DETAILS

Pet owner's username: Training

Type New Password:

ACCOUNT SETTINGS

Species: Dog

* Pet's Name: dillinger

* Pet owner's last name: test1

Mobile: +1 123456789

E-Mail Address: training@alrt.com

* Local time-zone: (GMT-05:00) Eastern Time (US + Canada)


* denotes required field.

SAVE CHANGES

8 Account settings

Click on the name in the top right corner at any time to view your account settings to:

- Change/verify your veterinarian login credentials
- Change/verify your veterinarian email address
- Reset veterinarian login password
- Add a new veterinarian to your practice portal

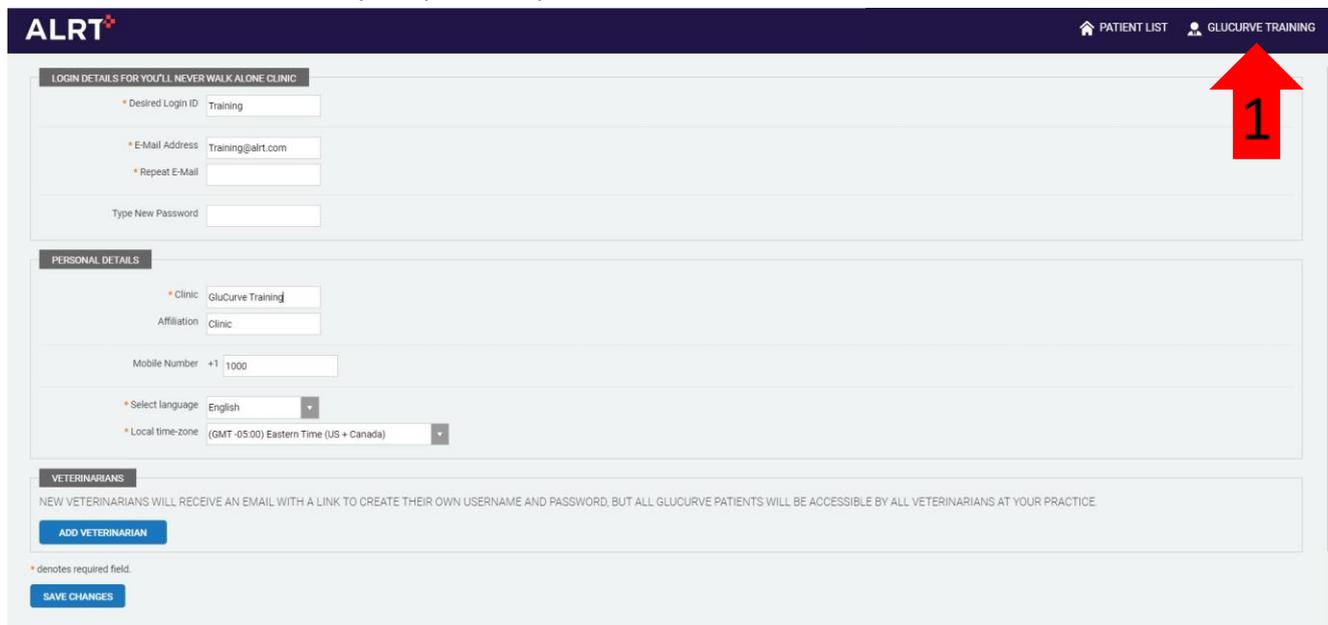

More information

Access the following resources for the GluCurve Pet CGM at www.GluCurve.Com/Help:

- [How to: Apply the GluCurve on a Pet](#)
- [How to: Activate the GluCurve after applying](#)
- [How to: Troubleshoot the GluCurve](#)
- [How to: Use the Veterinary Web Portal](#)
- [How to: Use the GluCurve app for iOS and Android](#)
- [Help Videos](#)
- [FAQs and More](#)