



How to: Use the GluCurve app

Notes

- Bluetooth must be enabled on the mobile device
- Android users must allow location service

1 Sign in

Open the GluCurve app for iOS or Android and sign in (if needed)

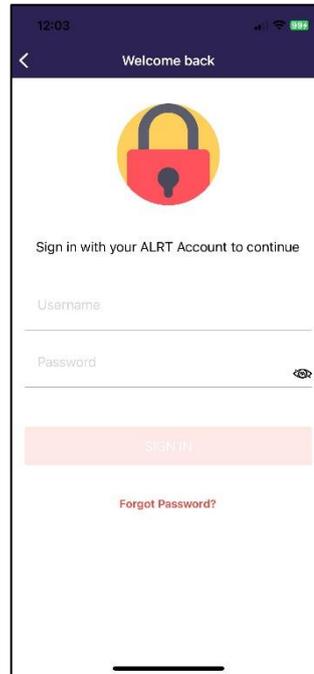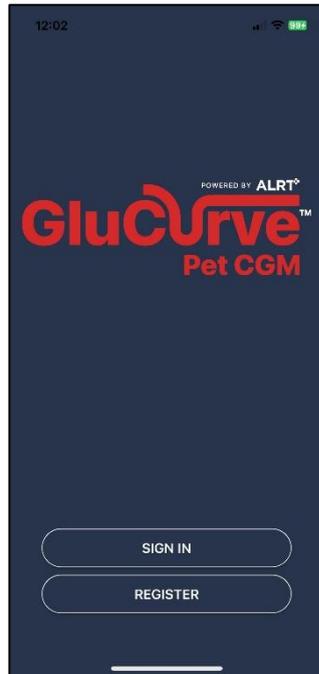

2 Main Screen – Glucose Data

Signing in opens to the main Glucose Data page where you can view live time and historic glucose readings.

The diagram illustrates the Glucose Data screen in two views: Graph and List View. Callouts point to various UI elements:


- Main Menu:** Points to the hamburger menu icon in the top left.
- Glucose Data:** Points to the graph area in the left view.
- Toggle Time of day:** Points to the 'Start' button and time selection controls at the bottom of the graph view.
- Toggle Between List / Graph:** Points to the 'View List' and 'View Graph' buttons in the top right.
- List View:** Points to the table view on the right.

TIME	BG (mg/dL)
05:51 PM	104
05:48 PM	107
05:45 PM	109
05:42 PM	109
05:39 PM	108
05:36 PM	107
05:33 PM	106
05:30 PM	107
05:27 PM	109
05:24 PM	108
05:21 PM	109
05:18 PM	107
05:15 PM	105

How to: Use the GluCurve app


3 Main Menu

The main menu gives you the following options:

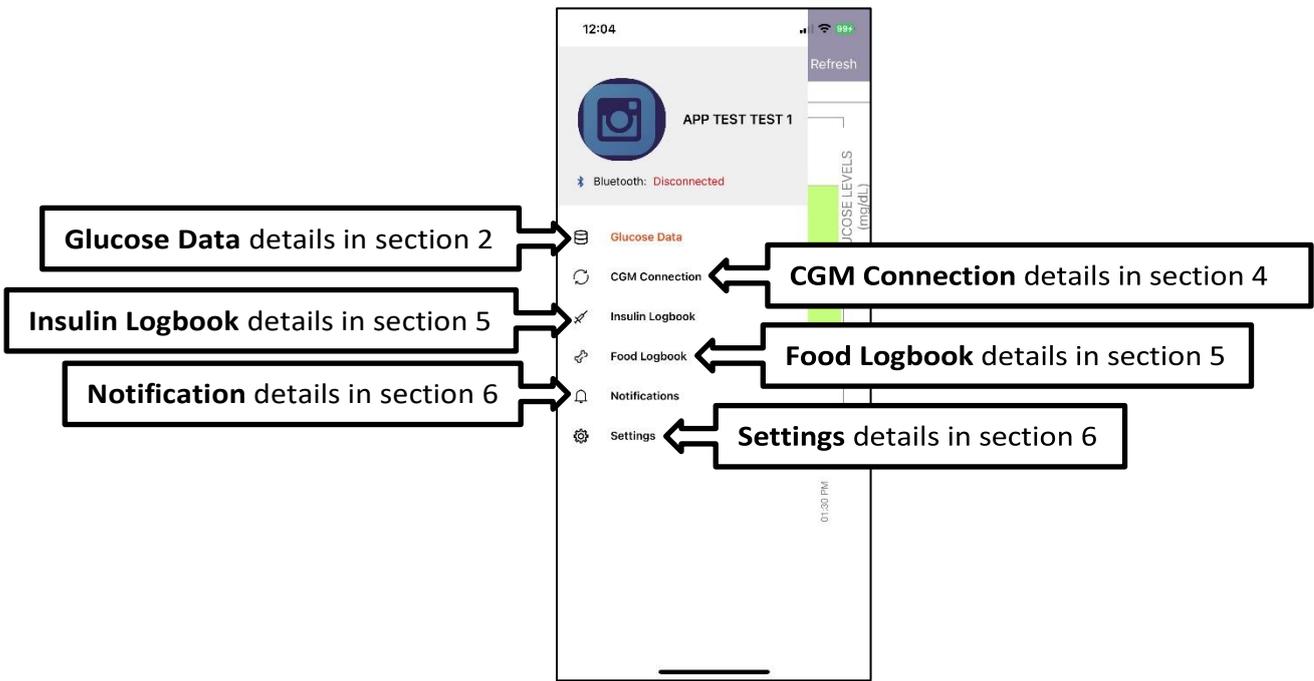

4 CGM Connection


CGM connection gives you the following information and options:

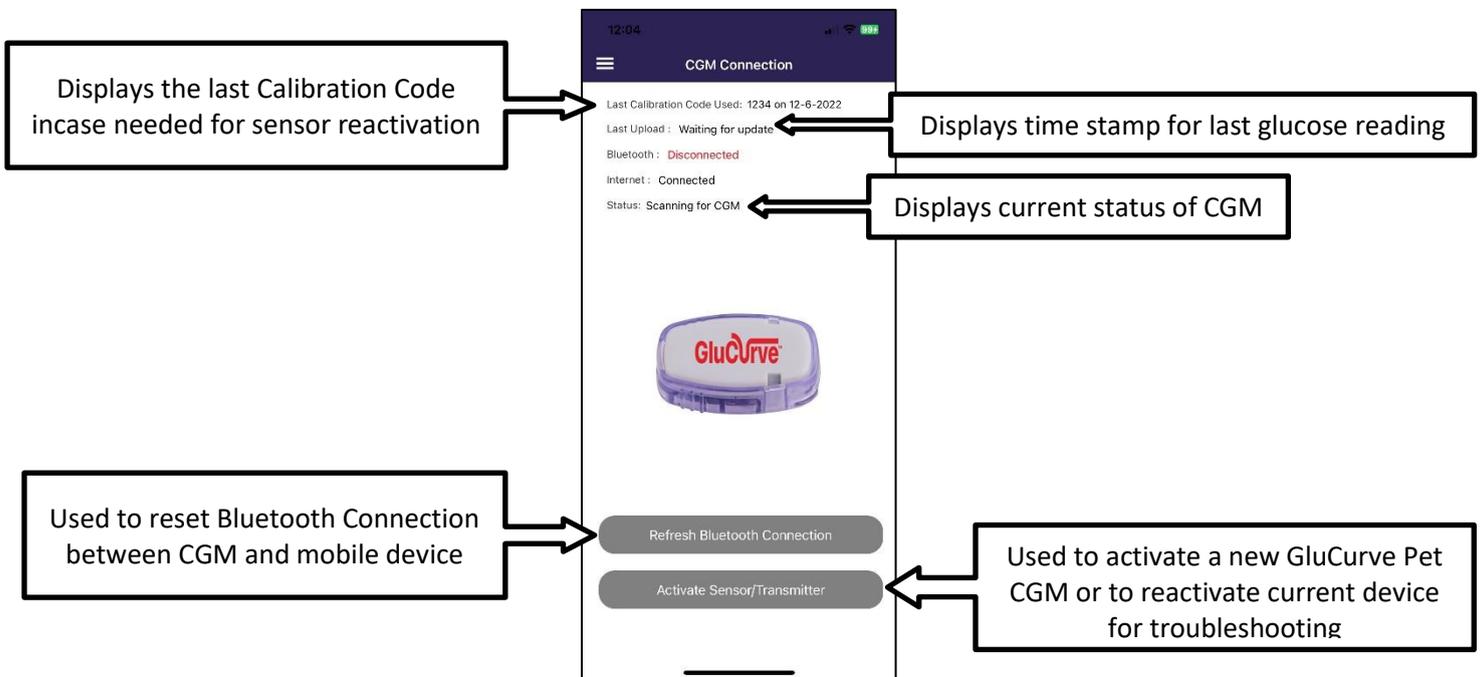

How to: Use the GluCurve app


5 Insulin and Food Logbook

The insulin logbook and food logbook should be filled out daily, the information is upload to the veterinarian.

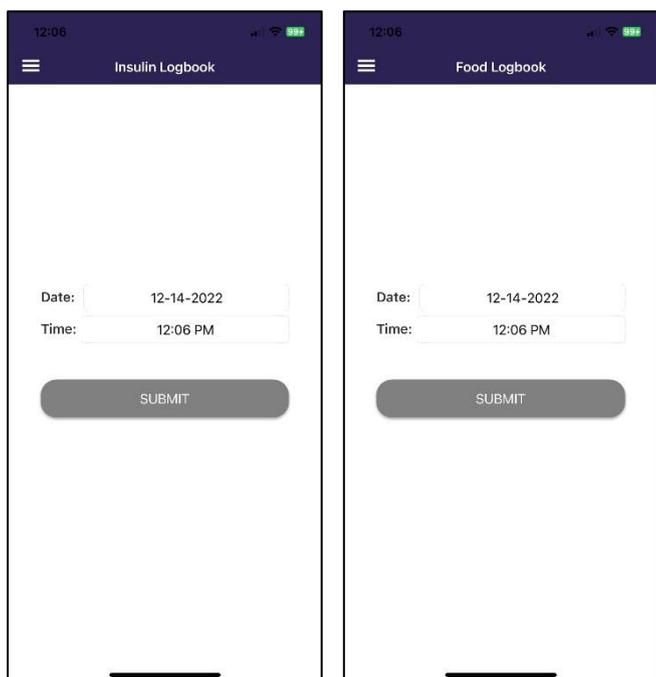

6 Notifications and Settings

Notifications is reserved for future use, Settings provides profile, device, app, and company info along with terms of use, privacy policy, an option to check for updates, and a calibration feature reserved for future use.

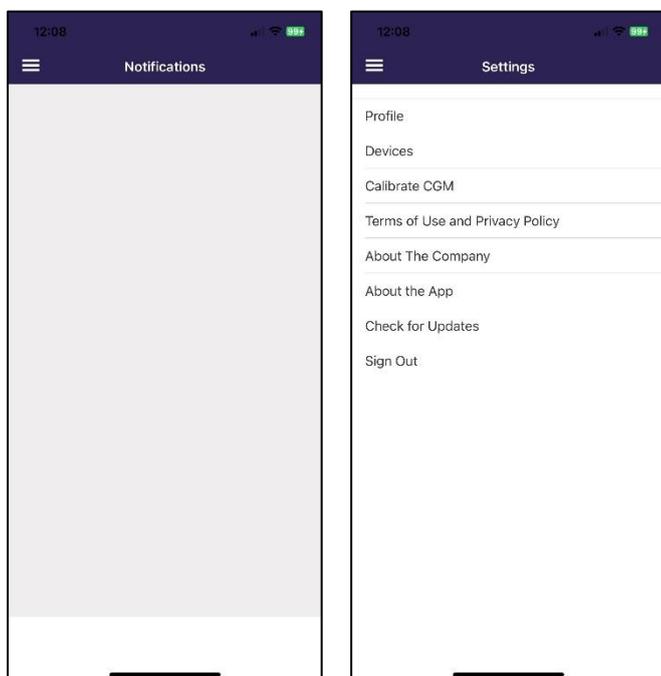

More information

Access the following resources for the GluCurve Pet CGM at www.GluCurve.Com/Help:

- [How to: Apply the GluCurve on a Pet](#)
- [How to: Activate the GluCurve after applying](#)
- [How to: Troubleshoot the GluCurve](#)
- [How to: Use the Veterinary Web Portal](#)
- [How to: Use the GluCurve app for iOS and Android](#)
- [Help Videos](#)
- [FAQs and More](#)