


How to: Activate the GluCurve after applying **ALRT**

Notes

- The **Calibration Code** on the Applicator package must be provided to the patient to activate
- Returning GluCurve patients can skip to step 5 of section 2 “Pet owner portion” since they are already registered in the Veterinary Portal and in the GluCurve App

1 Veterinarian portion

Go to www.GluCurve.com, returning veterinarians can enter username and password then click log in

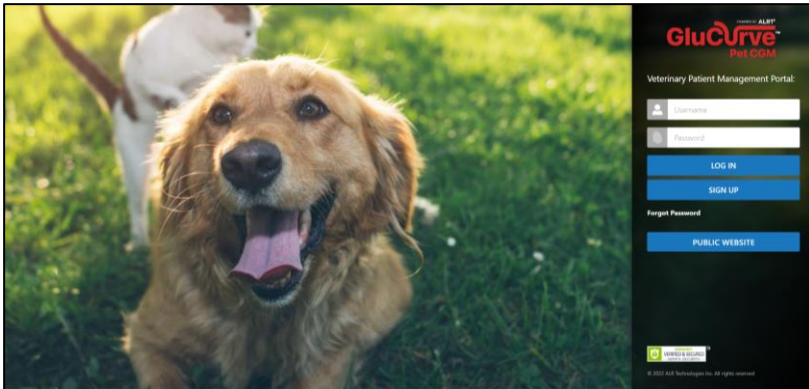

If this is your **clinic's first time** using a GluCurve Pet CGM, click sign up and create a free account

Click **add new patient** and fill out the patient registration form to add them to your Veterinarian Portal


PETS NAME	PET OWNER'S LAST NAME	LAST GLUCURVE CURVE	TYPE	ACTION
app test	test 1	1/18/2022 2:52:14 PM	Dog	
allegai	test 11	4/22/2022 2:50:02 PM	Cat	
jack	test10	6/23/2022 12:29:36 AM	Dog	
macho	test 8	3/8/2022 4:31:48 PM	Cat	
dinger	test1	3/4/2022 10:38:13 AM	Dog	
kenwood	test5	3/2/2022 10:38:05 AM	Cat	
sadie	test5	3/2/2022 8:33:37 AM	Cat	
diamond	test 4	3/1/2022 12:17:24 PM	Dog	
rustaf	test0	2/23/2022 5:30:37 PM	Dog	
goose	test7	2/19/2022 8:31:20 AM	Cat	
terran	test4	2/18/2022 10:14:28 PM	Cat	
gabnor	test2	2/18/2022 1:31:26 PM	Dog	
rustaf	test3		Dog (Pending registration)	

The pet owner will receive an email or text with instructions to pair bluetooth and complete activation, **they will need the Calibration Code from the Applicator package.**


How to: Activate the GluCurve after applying




2 Pet owner portion

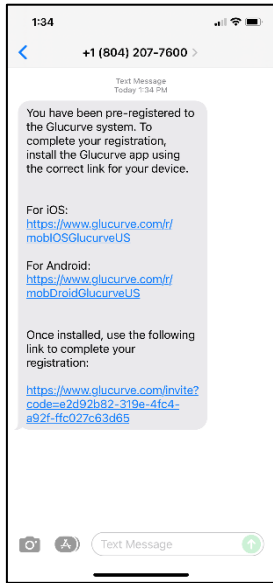

Pet owner will receive text/email with link to download the GluCurve App. Once downloaded, **ensure Bluetooth is enabled** then return to text/email and **click the registration link**.

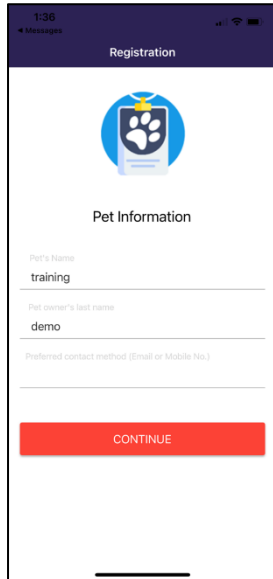


The registration link will take the pet owner into the GluCurve App to a "Pet Information" screen. Confirm information and click continue. If the **pet information screen isn't displayed**, hard close the GluCurve App and **re-click the registration link**

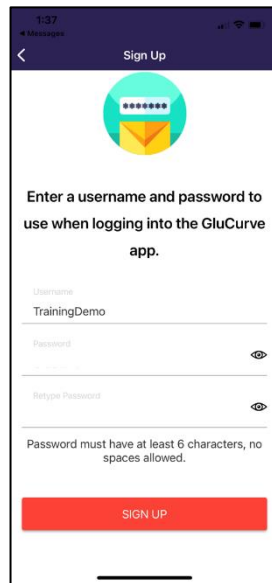


Pet owners will create a username and password that will be used for **this specific pet**.


Credentials may need to be shared with others.

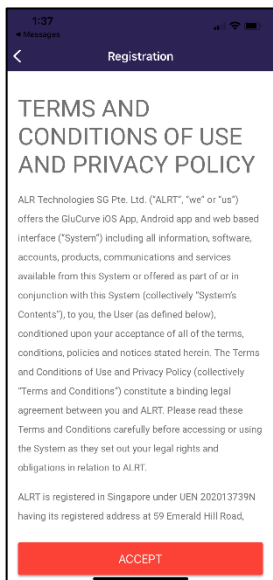

Pet owner will be asked to **accept** terms and conditions of use and privacy policy to continue.

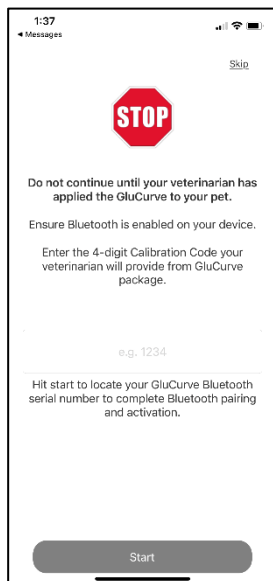


Provide 4-digit **Calibration Code** found on Applicator package and click **Start**. Must be **within 5-10 feet** of CGM. **Select** the Bluetooth Transmitter **serial number** located on the bottom of the GluCurve Pet CGM **box**.

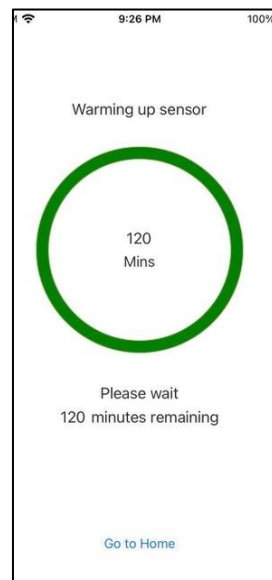

Activation is complete and the pet can be sent home. Glucose levels will start being displayed in 120 minutes.

More information

 Access the following resources for the GluCurve Pet CGM at www.GluCurve.Com/Help:

- [How to: Apply the GluCurve on a Pet](#)
- [How to: Activate the GluCurve after applying](#)
- [How to: Troubleshoot the GluCurve](#)
- [How to: Use the Veterinary Web Portal](#)
- [How to: Use the GluCurve app for iOS and Android](#)
- [Help Videos](#)
- [FAQs and More](#)